



UPEASY – MANUAL

INSTRUCTIONS

We recommend that you use this cushion on a sturdy chair with sides, and not on wheelchairs.

- Push down on the blue part of the cushion, pulling it away from the bottom part.
- Select a weight setting by lifting the mechanism pin and entering it into the groove of the appropriate weight.

- Place the Upeasy on the chair and sit down.

Note: If the cushion does not go down when you sit on it, re-adjust the weight setting to a lower weight.

- When you are ready to get up, lean forward and the UpEasy will rise, to help you get up.

Kindly return all equipment to Ezra Umarpeh in a clean condition. You will find the 'due back date' printed on the loan agreement. If this item is needed for a longer period of time, please contact us.